Accu-Chek Insight Flex infusion set

The Accu-Chek Insight Flex infusion set is intended for the subcutaneous infusion of insulin with an Accu-Chek Insight insulin pump. The Accu-Chek Insight Flex infusion set consists of the head set (A–G) and the transfer set (H).

- Package 1 contents (with transfer set):
- Head set (A–G)
 - Transfer set (H)

- Package 2 contents (without transfer set):
- Head set (A–G)

A Adhesive plaster	Canula			
B Adhesive liner	Length: 6 mm	8 mm	10 mm	
C Connector plate	Fill amount 0.7 µl	0.7 µl	0.7 µl	
D Introducer needle	(approx.): 7 µl	7 µl	7 µl	
E Needle box				
F Button				
G Disconnect cover				
H Transfer set (tubing)	Length: 40 cm	70 cm	100 cm	
	Fill amount 9 µl	13 µl	17 µl	
	(approx.): 90 µl	130 µl	170 µl	

* U100 insulin

- Before unpacking your infusion set, wash your hands thoroughly. Choose and disinfect your infusion site as instructed by your doctor or healthcare team.
- Remove the white liner on top (A), which will expose the blue liner. Then remove the blue liner (B).
- Push the button until it stops. The introducer needle with the soft canula snaps out.
- Insert the infusion set at a 90° angle. Choose an infusion site away from your waistline, bones, any scar tissue, belly button and recent infusion sites. Rotate your infusion site regularly.
- Press the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.
- Remove the needle box with the introducer needle at a right angle from the body, leaving the adhesive plaster and the connector plate in place on your skin. The introducer needle snaps back into the needle box automatically.
- Attach the transfer set to the insulin pump and fill it according to the instructions provided with the insulin pump. Make sure it is completely filled and free from air bubbles.
- Connect the filled transfer set to the connector plate. Before resuming insulin pump therapy, fill the head set and the soft canula with 0.7 U of insulin.

In US, distributed by:
Roche Diabetes Care, Inc.
Indianapolis, IN 46256-USA
Tel: 1-800-250-7801
ACCU-CHEK Customer Care Service Center: 1-800-688-4578
www.accu-chek.com

To disconnect the set safely, squeeze the side wings of the connector together and withdraw it. Keep the connector plate clean. Attach the disconnect cover when appropriate.

Warning

- When using the infusion set for the first time, your doctor or healthcare team must be present.
- Protect the product from excessive humidity, sunlight and heat. Store at room temperature.
- Check your blood glucose level at least 4 times a day, and at least once, 1 to 3 hours after inserting a new infusion set.
- Do not use this product for any medication other than insulin.
- Do not use this product if the blister package is already opened or damaged.
- If the infusion site becomes inflamed, replace your infusion set at a new site.
- Change the head set at least every 3 days and the transfer set at least every 6 days, or as instructed by your doctor or healthcare team. Also consult the instructions for use of the insulin pump.
- Discard your infusion set after use, according to local regulations. Do not re-use it to avoid infection and underdelivery of insulin.
- Dispose of your infusion set and needle box safely, so that no one can be pricked or hurt by it, as this could lead to infection. Regularly check that the infusion set is properly attached to the skin to prevent accidental removal of insulin.
- If your blood glucose level should suddenly increase, or if an occlusion occurs, check for clogging or leaks. Change your infusion set immediately if you are not entirely sure that it is working properly.

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Accu-Chek Insight Flex Infusionsset

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