Accu-Chek Insight Tender infusion set

The Accu-Chek Insight Tender infusion set is intended for the subcutaneous infusion of insulin with an Accu-Chek Insight insulin pump. The Accu-Chek Insight Tender infusion set consists of the head set (A-H) and the transfer set (I).

Package 1 contents (with transfer set):

- Head set (A-H)
- Transfer set (I)

Package 2 contents (without transfer set):

- Head set (A-H)

	Cannula Length: Fill amount (approx.):	13 mm 0.9 U* 9 µl	17 mm 0.9 U* 9 µl
A Needle cover			
B Handle			
C Introducer needle			
D Connector plate			
E Adhesive plaster			
F Rear adhesive liner			
G Front adhesive liner			
H Disconnect cover			
I Transfer set (tubing)			

*1000 insulin

- Before unpacking your infusion set, wash your hands thoroughly. Choose and disinfect your infusion site as instructed by your doctor or healthcare team. Pull back the needle cover until you hear an audible **CLICK**. This indicates that the needle cover is in place.
- Insert the infusion set at a 20° to 45° angle. Choose an infusion site away from your waistline, bones, any scar tissue, belly button and recent infusion sites. Rotate your infusion site regularly.
- Carefully peel the front adhesive liner away from the adhesive plaster.
- Press the front part of the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.
- While holding the adhesive plaster in place, pull the handle with the introducer needle sideways out of the connector plate.
- Push the needle cover back until it snaps into its original position and dispose of it.
- Peel the rear adhesive liner away from the adhesive plaster.
- Press the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.
- Attach the transfer set to the insulin pump and fill it according to the instructions provided with the insulin pump. Make sure it is completely filled and free from air bubbles.
- Connect the filled transfer set to the connector plate. Before resuming insulin pump therapy, fill the head set and the soft cannula with 0.9 U of insulin.

To disconnect the transfer set, squeeze the side wings of the connector together and withdraw it. Keep the connector plate clean. Attach the disconnect cover when appropriate.

Warning

- When using the infusion set for the first time, your doctor or healthcare team must be present.
- Check your blood glucose level at least 4 times a day, and at least once, 1 to 3 hours after inserting a new infusion set.
- If your blood glucose level should unexpectedly increase, or if an occlusion occurs, check for clogging or leaks. Change your infusion set immediately if you are not entirely sure that it is working properly.
- If the infusion site becomes inflamed, replace your infusion set at a new site.
- Change the head set at least every 3 days and the transfer set at least every 6 days, or as instructed by your doctor or healthcare team.
- Regularly check that your infusion set is properly attached to the skin to prevent accidental underdelivery of insulin.

Accu-Chek Insight Tender Infusionsset

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© 2017 Roche Diabetes Care