ar

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en

- Refer to the list of compatible pens available on the box.
- Do not use if package is damaged.
- Refer to pen injector instructions for use.
- Refer to the pen needle size on the box.
- Reuse can dull the needle and be linked to hardening of fat under the skin. This may change insulin absorption, making blood glucose control more difficult¹.

Consult your healthcare professional for the needle length and the injection technique most appropriate for you.

To ensure an accurate dose when injecting with an insulin pen:

- Check the flow of insulin through the needle after loading a new cartridge and before every injection.
- Attach a new needle before each injection and remove it immediately after the injection. Never carry a pen with a needle attached as this leaves an open passageway into the cartridge.
- Insert the needle into the skin in the way that your healthcare professional suggests. After needle insertion, push the injection button in as far as it will go and continue to press firmly for at least ten seconds before removing the needle from the skin. If insulin continues to drip from the needle after injection, hold the needle in the skin longer the next time you inject.

¹Frid AH, Kreugel G, Grassi G, et al. New insulin delivery recommendations. Mayo Clin Proc. September 2016;91(9):1231-1255

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© 2019 Roche.
ACCU-CHEK and ACCU-FINE are trademarks of Roche.